EN MICRO TOOLS

ICSI INJECTION PIPETTES

Micro Tools for ICSI Injection (MT-INJ): 10 pcs/box

INDICATION FOR USE

The Intracytoplasmic Sperm injection (MT-ICSI) Injection Pipettes are used to inject a single sperm into oocyte.

INSTRUCTIONS FOR USE

Preparation

- Make the drop of culture medium and PVP, cover with Hypure Oil. Equilibrate it in the incubator for 15 minutes.
- Set the pipette to suitable manipulation holder.
- Note: Recommend choosing the product whose inner diameter matches diameter of sperm in order to prevent a large amount injection of PVP.
- Place oocyte in culture medium, place motile sperm in PVP.

Instructions for use

1. Aspirate sperm tail-first into MT-INJ.
2. Move the polar body at the 12 or 6 o'clock position, and hold oocyte with MT-HD.
3. Move sperm to tip of MT-INJ.
4. Penetrate oocyte with MT-INJ.

Note: Make sure that the cell membrane is ruptured.

- 5. Inject single sperm into oocyte with minute amounts of PVP.
- 6. Gently withdraw MT-INJ.

HOLDING PIPETTE

Micro Tools for ICSI Holding (MT-HD): 10 pcs/box

INDICATION FOR USE

The Holding pipettes (MT-HD) are used to hold the oocyte or embryo in position with the application of vacuum during the injection of a single sperm with the ICSI injection pipette, and other micromanipulation procedures.

INSTRUCTIONS FOR USE

Preparation

• Set the pipette to suitable manipulation holder.

Instructions for use

1. Hold oocyte or embryo with MT-HD.
2. Keep holding it until the manipulation is finished.

BIOPSY PIPETTES

Micro Tools for Polar Body Biopsy (MT-BPPD): 10 pcs/box. Micro Tools for Blastomere Biopsy (MT-BPBM): 10 pcs/box.

INDICATION FOR USE

The Polar Body Biopsy pipettes (MT-BPPD) are tools used in IVF/Assisted Reproduction Technology (ART) laboratories for removal of polar bodies from oocytes, which may be done in order to perform pre-implantation genetic diagnosis on the genetic material in the biopsied cell(s).

The Blastomere Biopsy Pipettes (MT-BPBM) are tools used in IVF/Assisted Reproduction Technology (ART) laboratories for removal of blastomere(s) form embryos, which may be done in order to perform pre-implantation genetic diagnosis on the genetic material in biopsied cell(s).

INSTRUCTIONS FOR USE

- Preparation - Make the drop of culture medium, cover with Hypure Oil. Equilibrate it in the incubator for 15 minutes.
- Set the pipette to suitable manipulation holder.
- Place oocyte or embryo in culture medium.

Instructions for use

1. Open the zona pellucida, and then extract polar body from oocyte, blastomere from embryo or trophoctoderm from blastocyst.
2. Use the extracted cell and perform genetic diagnosis.

PARTIAL ZONE DISSECTION PIPETTE

Micro Tools for PZD (MT-PZD): 10 pcs/box

INDICATION FOR USE

The Partial Zone Dissection (MT-PZD) are used to cut an opening on the zone of an embryo mechanically in assisting embryo hatching prior to implantation.

INSTRUCTIONS FOR USE

Preparation

- Make the drop of culture medium, cover with Hypure Oil. Equilibrate it in the incubator for 15 minutes.
- Set the pipette to suitable manipulation holder.
- Place embryo in culture medium.

Instructions for use (Cross dissection process of zona pellucida):

1. Cut a part of zona pellucida in the cross shape with MT-PZD. Note: Do not allow pipette to touch blastomere at this time. This could damage embryo.

GENERAL INFORMATION

Raw Material

Borosilicate glass

Quality Control Specification

Each lot of Micro Tools receives the following tests

- Endotoxin (≤ 0.5 EU/device (LAL)
- Mouse Embryo Assay (2-cell MEA test :≥80%)

Storage instructions, and stability

- Keep away from sunlight. • Keep dry.

Contraindications

Note: Confirm specification of the product in the labeling prior use

- To avoid deterioration, do not re-sterilize.
- Do not re-use. Reuse may cause contamination.
- Do not use if the package is opened or damaged.
- Do not use the product after expiration date or infected product and discard it (see the warning).

Warnings - Aseptic technique should be used.

Kitazato Corporation Shizuoka Office
81 Nakajima, Fuji, Shizuoka 416-0907 Japan
TEL +81-545-66-2202 FAX: +81-545-60-5772

• Use sterilized equipment and materials only.

• The product is sterile until the expiration date shown on the label if the package is unopened or undamaged
• In case of infection, dispose the product appropriately in a prescribed manner.

- The user shall be responsible for any problems caused by non-conformity to the present IFU.
- Observe all federal, state and local environmental regulations when discarding the product.
- The detailed usage of Micro Tools are determined based on each laboratory's procedures and protocols that are optimized for individual medical purposes.
- This product is intended to be used by medical specialists trained in fertility treatment.

Note: Single use device. Lifetime of the device is determined by the applied AR technique and is maximum to the period of micromanipulation of human gametes or embryos. The user is warned to follow safe, established laboratory practices in order to prevent accidental DNA contamination after the micropipette has been from its protective covering.

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